


Month:	 <b>Medonic M Series Cleaning Maintenance - Log Form</b>																															
Year:																																
Frequency	Day of the Month →	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
Daily Cleaning	Clean OT Aspiration Probe	✓																														
	Clean Capillary Adaptor weekly	✓																														
	Check Reagents weekly	✓																														
	General Cleaning	✓																														
<b>Lab. Tech. Initials: →</b>																																
If you run less than 40 Samples a Day perform this maintenance once a Month												If you run more than 50 Samples a Day perform Monthly maintenance bi-weekly																				
Monthly Cleaning - See Monthly Cleaning Flowchart	Clean Top white washer above OT probe	✓	Date Performed & Lab Tech Name										✓	Clean Top white washer above OT probe	Date Performed & Lab Tech Name																	
	Clean OT Aspiration Probe	✓											✓	Clean OT Aspiration Probe																		
	Clean Probe Wash Station-CP	✓											✓	Clean Probe Wash Station-CP																		
	Aspirate <b>Sol. No.2</b> in PD	✓											✓	Aspirate <b>Sol. No.2</b> in PD																		
	Aspirate Diluent. in PD	✓											✓	Aspirate Diluent. in PD																		
	Preform Clean Orifice x 3	✓											✓	Preform Clean Orifice x 3																		
	<b>Perform Clot Prevention</b>	✓											✓	<b>Perform Clot Prevention</b>																		
	Run Background	✓											✓	Run Background																		
<b>IMPORTANT NOTE → Perform 3 Month Maintenance Cleaning "before" you Calibrate Analyzer ← IMPORTANT NOTE</b>																																
3 Month Cleaning - See 3 month Cleaning Flowchart	<b>Solution Bottle No.1</b> (Blue/Purple Solution) <b>Enzymatic Cleaner</b>	✓	Tech Name										Notes / Observations										Acceptable Background Ranges									
	<b>Solution Bottle No.2</b> <b>Hypochlorite 2%</b> (Light Green Solution)	✓																					WBC ≤ 0.1									
	<b>Solution Bottle No.3</b> <b>Detergent Cleaner</b> (Pink Solution)	✓																					RBC ≤ 0.01									
																							HGB ≤ 0.2									
																							PLT ≤ 10									
																							Boule Cleaning Kit Part No. CDS 501-036									
Calibrator CDS 501-606 Lot No.:																																
<b>Next Calibration Due Date:</b>																																
Date																																
Tech:																																
QC Log	QC Kit Lot #		First Set of Controls.					Second Set of Controls.					NOTES:.																			
			Open Date		14 Day Exp. Date			Open Date		14 Day Exp. Date																						
QC Controls	Low Ctrl.	00000-00	MM / DD / YYYY		MM / DD / YYYY			MM / DD / YYYY		MM / DD / YYYY																						
	Nor. Ctrl.	00000-01																														
	High Ctrl.	00000-02																														