

STANDBY

Main Menu

Prime System Reagent Setup

Power Down **Standby**

Advanced Q/C

Dispense

Sample List Menu

Print **New Sample**

The system is filled with liquid and is prepared for power off. Press [PWR UP] if you want to return the system to active status or [EXIT] if you want to return to instrument menu. It is recommended to use [ENTER STANDBY] and that power is left on, instead of using this feature.

The system has not been used during the preset display saver time. Press [RESUME] to activate the instrument. Once activated, the instrument is ready to perform an analysis.

Main Menu

Standby

Going to Standby.

Please wait...

Ok Cancel

Sample List Menu

Print New Sample

Standby Mode

System is in Standby.

Exit Standby

Serial no: 4711
Firmware: 2.9.3

Sample List Menu

Print New Sample Sampling Device

The instrument is in the process of going into Standby. Please wait.

The system is in Standby. Press [EXIT STANDBY] to activate the instrument. Once activated, the instrument is ready to perform an analysis.

Standby, Power Down, and Power Up Informational Displays

Exit Standby

Now exiting Standby Mode.

Please wait...

Cancel

Sample List Menu

Print New Sample

Main Menu

Power Down

Power Down in progress.

Please wait...

Sample List Menu

Print New Sample

Power Up

Power up in progress.

Please wait...

Cancel

Sample List Menu

Print New Sample