


How to Set Date / Time on the Medonic M Series Analyzer

Instruments Included
 Medonic Open Vial – 1400073; 1400106
 Medonic Cap Pierce – 1400074
 Medonic Autoloader - 1400075

To set both Date and Time, it is a simple three (3) step process. However, the date or the time can also be set individually using the instructions below. In addition, the date format can be changed if needed. See date format options listed in Step 2 below.

Additional information can be located in the User Manual – Section 4 – Instrument Setup.

STEP 1

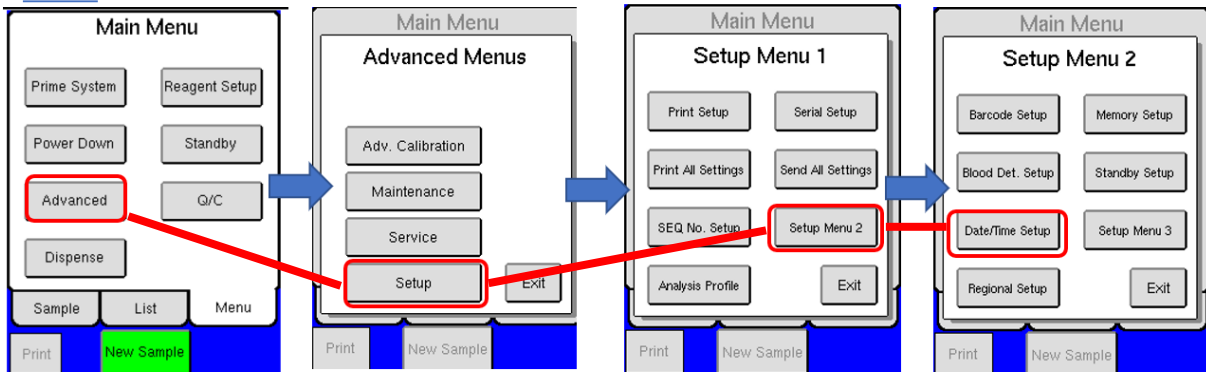


From Main Menu – Press [Advanced] [Setup] [Setup Menu 2] [Date/Time Setup]

Note – depending on current instrument software level, the **Advanced Menu** may be slightly different, however the [Setup] button is always present regardless of software level.

To Set the Date or Date Format

STEP 2

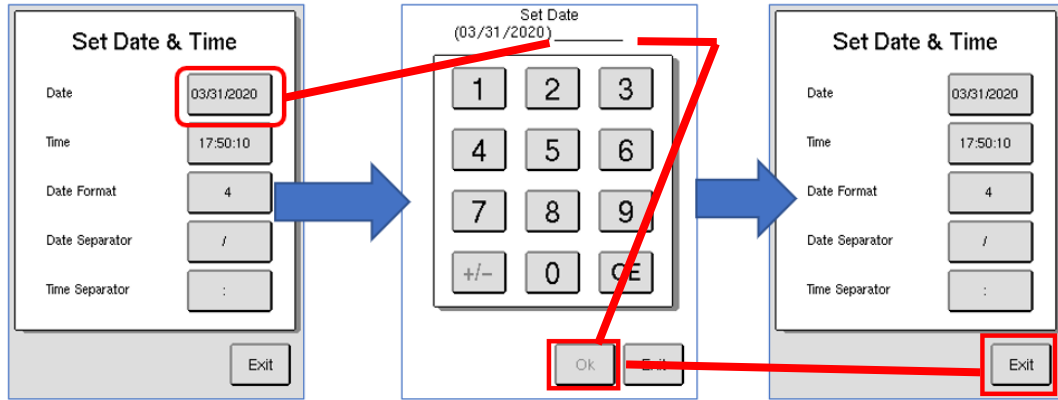

Press [Date] Button Enter Date and Press [OK] Press [EXIT] Button

Options for Date format are shown below


<u>Option</u>	<u>Date Format</u>
1	DD/MM/YY
2	YY/MM/DD
3	YY/DD/MM
4	MM/DD/YY

To Set the Time

The time function uses a 24 hour clock. The time is shown / entered from 00:00 to 24:00.

Example: 5:00 AM is shown as 05:00 and 5:00 PM is shown as 17:00.

STEP 3

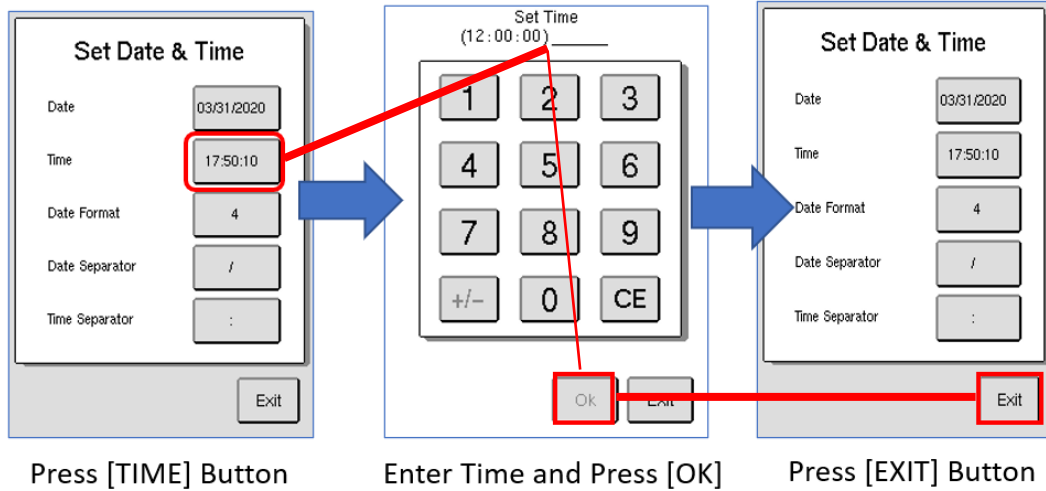

Reference Information

- User Manual – Section 4 – Instrument Setup
- Technical Support
 - TechSupport@boule.com
 - 954.791.1773 Option 2