

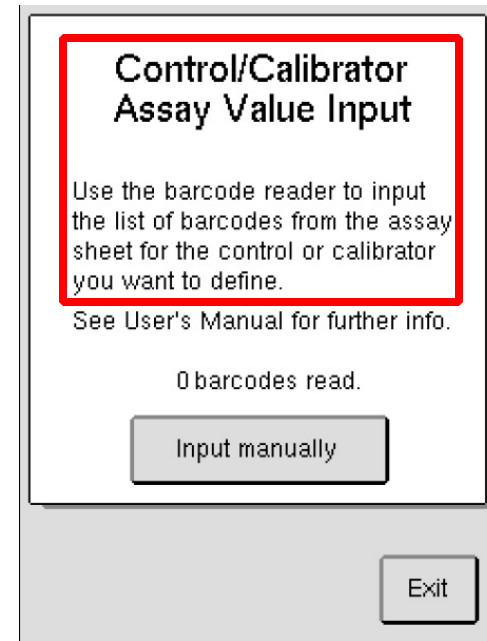
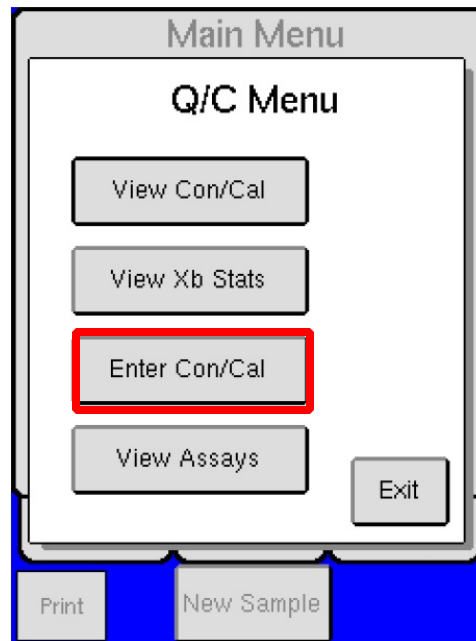
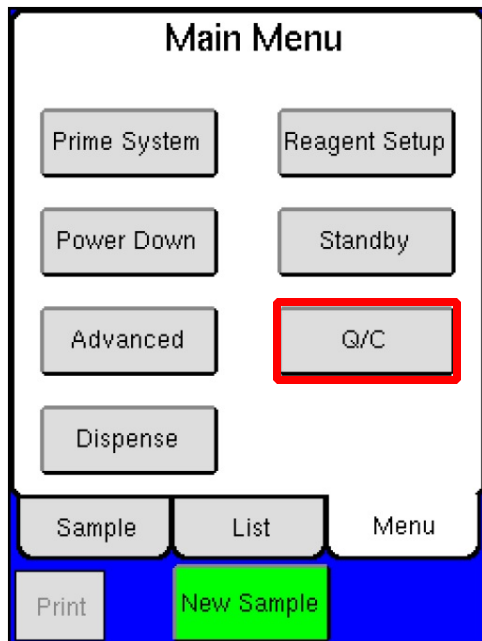
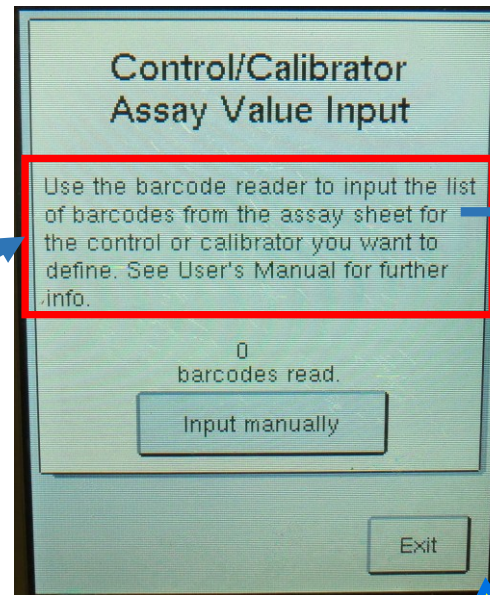


## Enter New Lot of QC Control

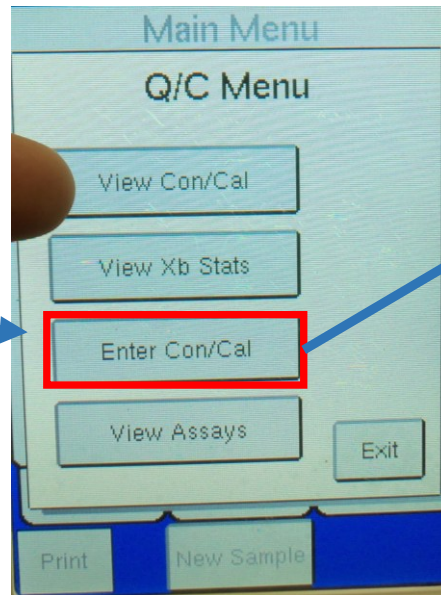

# Enter New Lot QC Controls

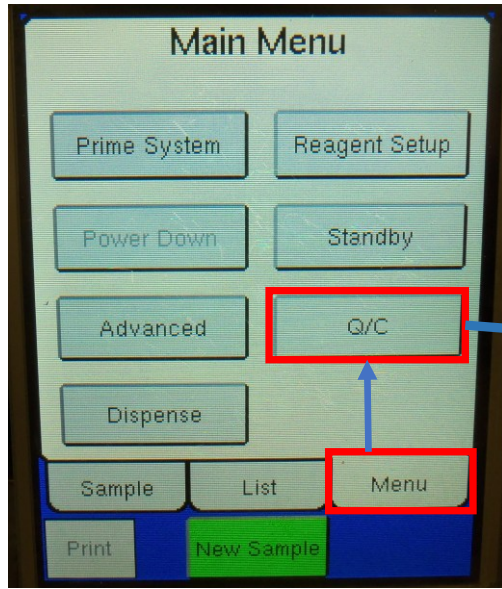

- For Medonic M-series with barcode reader**
1. From the Main Menu go to [Q/C Menu] [Enter Con/Cal] [Control/Calibrator Assay Value Input]
  2. If insufficient space message is displayed, follow screen instruction to delete oldest lot.
  3. Scan barcodes in order from 1 to 9 from the left most column – do not skip a barcode.
  4. Once complete, exit the Control / Calibrator Assay Value Input screen and return to sample run screen.
  5. Scan Low level blood control vial barcode and analyze control as per User Manual.
  6. Repeat steps 1-4 using barcodes from center column.
  7. Repeat step 5 using Normal level blood control.
  8. Repeat steps 1-4 using barcodes from right most column, then repeat step 5 using High level blood control.

Instr.: M-series	Instr.: M-series	Instr.: M-series
Lot: 22603-01	Lot: 22603-02	Lot: 22603-03
Prod.: Con-Diff Low	Prod.: Con-Diff Norm	Prod.: Con-Diff High
ID: 2260301+	ID: 2260302+	ID: 2260303+
Exp.: 27-Jul-2026	Exp.: 27-Jul-2026	Exp.: 7-Aug-2026

