

Medonic M Series 3 Month Maintenance Cleaning

SOLUTIONS NEEDED: Boule Cleaning Kit P/N: CDS 501- 036

Solution Bottle #1: Enzymatic Cleaner (Blue), Solution Bottle #2: Hypochlorite Cleaner (Clear Green), Solution Bottle #3: Detergent Cleaner (Pink)

STEP 1 Remove Diluent (RED) & Lytic Pickup (Yellow) Tubes from Reagent Pack container.
Then, Press [MENU-ADVANCE - MAINTENANCE - CLEANING MENU]

STEP 2 Press [CLEAN CYCLE EMPTY]. Wait 4 minutes to empty.

Bottle No.1 (Blue/purple Solution)

STEP 3 Insert Diluent & Lytic Pickup Tubes in Bottle No.1 (Blue/purple Solution) and press [CLEAN CYCLE FILL]. Wait 4 min for cycle to finish. **Remove pick up tubes from Bottle No. 1.** Wipe pickup tubes with paper towel, and leave outside of Reagent pack. Set your timer to 30 min. **Wait 30 min. to soak and clean.**

STEP 4 After 30 min. Press [CLEAN CYCLE EMPTY]. to empty solution #1 blue solution.
Wait 4 minutes to empty.

Bottle No.2 (Light Green Solution)

STEP 5 Insert Diluent & Lytic Pickup Tubes in Bottle No.2 (Light Green Hypochlorite Solution) and press [CLEAN CYCLE FILL]. Wait 4 min for aspiration cycle to finish. **Remove pick up tubes from Bottle No.2.** Wipe pickup tubes with paper towel, and leave outside of Reagent pack. Set your timer to 20 min. **Wait 20 min. to soak and clean.**

STEP 6 After 20 min. Press [CLEAN CYCLE EMPTY]. to empty Solution #2 (light green solution)
Wait 4 minutes to empty.

Bottle No. 3 (Pink Solution)

STEP 7 Insert Diluent & Lytic Pickup Tubes in Bottle No.3 (Detergent - Pink Solution) then press [CLEAN CYCLE FILL]. Wait 4 min for aspiration cycle to finish. **Remove pick up tubes from Bottle No.3.** Wipe pickup tubes with paper towel, and leave outside of Reagent pack. Set your timer to 30 min. **Wait 30 min. to soak and clean.**

STEP 8 After 30 min. Press [CLEAN CYCLE EMPTY]. to empty pink solution.
Wait 4 minutes to empty.

STEP 9 Grab pickup tubes and **wipe tubes with a towel dampened with Diluent.** Next insert **Diluent Pickup tube (Red)** in Diluent container & **Lytic Pickup (Yellow)** tube in Lytic container. Then press [CLEAN CYCLE FILL]. Wait 4 minutes until completed.

STEP 10 Press Menu, Select [ADVANCE - SERVICE- INSTRUMENT LOG - Press Print]
Keep instrument log report with your maintenance record.

STEP 11 Perform a background count 3 times. Run an old sample or Normal Control to prime system then run your next next sample.

END